CP0277-001

Manufacturer: INTERIO JO INC. 28825, Sandan-ro 15beon-3, Gyeonggi-do, 17444, Korea

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- \*Betekenis van de symbolen 1. Manufacturer 2. USE BY 3. EC REP 4. EU representative 5. Sterilized 6. Sterile using steam or dry heat 7. Caution 8. Instructions for use 9. CE 10. CE Marking 11. CAUTION 12. Date of manufacture 13. LOT 14. Batch code 15. Temperature limit

- EN Purpose of Use : Vision Correction The product herein described is a medical device How to Store before use : Stored at room temperature \*How to Use 1. Precautions Before Use 1) Strictly follow instructions of the user manual or from specialists. 2) Be sure to wash your hands before wearing or taking off lenses. 3) Be careful your nails do damage to lenses. 4) Check whether lenses are upside down while wearing 5) Be careful that cosmetics, lotion, soap, cream, deodorant do not contact your eyes or lenses.

- 6) Identifying the front and the back of the soft lenses. 7. How to Wear and Take off lenses 1) How to wear lenses

Washing	Put the lens on your hand and drop two or three drops of the cleaning solution, and washing the lens rubbing gently.
Rinsing	After washing, rinse the lenses with the managing solution, and put the lenses in their right or left side of the case with the storing solution.

Symptoms	Causes	Treatments
Lots of tears of flow. Vision is blurred. Lenses turn easily. A little bit of irritation. Feeling a bit of congestion. Lots of tears flow and irritation is too severe.	These are natural symptoms occurring after wearing lenses. Its improper washing. Foreign substances have deposited on the lenses due to too long duration of wearing lenses. Either lenses or eyes are problematic.	With individual differences, users can get used to them. If you continue to bother, you'd better consult a specialist. Wash lenses with cleaning solution carefully and rinse them completely with storing solution before use. Consult your doctor immediately and follow their instructions.

Symptoms	Causes	Treatments
Blurry vision	Foreign substances are deposited on the lenses.	Wash the lenses with the cleaning solution, and if the symptoms are severe, remove the foreign substances with protein removing solution.
Blurry vision	Lenses have foreign substances, dust, or scratches.	Wash the lenses and consult a specialist
Blurry vision	Fat or protein has been deposited on the lenses or the duration of wearing is too long.	Wash the lenses carefully. Remove the lenses and take a rest.

Sterilizing (Chemical)	Fill the case with the storing solution and store it for the time which is specified by the manufacturer.
Rinsing	After washing, rinse the lenses with the managing solution, and put the lenses in their right or left side of the case with the storing solution.

- 4. Purpose of Use and Effect and Efficacy It is a unifocal lens which is attached directly to the eyeball and corrects the vision. \*Precautions for Use 1. Do not use if you have diseases or are allergic to contact lenses. 2. Check whether you have a systemic disease or eye diseases, consult your doctor and follow their instructions. 3. If you have a severe pain or experience irritation during use, stop using them, and consult your doctor. 4. If the vision has been made unstable due to dryness of the surface of the lenses, apply artificial tear solution and blink several times, which could solve the trouble. 5. If you work, lack of sleep, long reading, etc. have made your eyes sore, try to stop wearing lenses at least a rest. 6. If you use soap, shampoo, spray, and cosmetics, be careful that they don't contact the lenses, and make up should go after wearing lenses. 7. When swimming, washing, rinsing, or lubricating, use the proper solution for the purpose. 8. When using physiological saline solution, cleaning solution and storing solution, be sure to use the products which are manufactured by the proper specialty companies, and never use tap water, boiled water or other improper products. 9. Store contact lenses in the dedicated storing solution. 10. Be careful that your fingernails do not cause damage to the lenses. 11. Wash your hands before wearing lenses. 12. Because contact lenses can be cleaned with paper, fabrics, or glass cloth. 13. When swimming or taking a shower, be sure to remove lenses and wear swimming goggles. 14. Except the eyes and artificial lenses, contact lenses for therapy purposes cannot be used with the lens. 15. Do not exceed the expiration date of the lenses and do not use broken lenses. 16. Do not use the lenses another has, be careful not to change the lens left and right if you have different power of lenses on the each eye. 17. When you wear contact lenses, be careful not to change the lens left and right if you have different power of the lenses continue to bother with all that, then consult your doctor.

- \*Precautions for side-effects If you experience the problems below, check the precautions. 1. Irritation, burning, itching, soreness, and other eye diseases in the eyes. 2. Feeling irritation from the lenses. 3. The eyes are itchy and have pain. 4. Eye diseases such as conjunctivitis, keratitis. 5. Too much tears. 6. Abnormal amount of secretion. 7. Congestion and dryness of the eyes. 8. Narrowed or impaired vision. 9. Halos of the object. 10. Turning lenses due to wearing them off the axis (Toric) 11. Lenses turn and move off the axis (Toric) [Corrective Measures] 1. Remove the lens. If the trouble disappears, check the lenses and, even if there appears no problem, consult your doctor before using them. 2. If the lenses are contaminated, wash the lenses and put them on again. And if the same problem occurs again, then consult your doctor. \*Prohibition Those who have the symptoms listed below should not use lenses. 1. Acute or semi-chronic inflammation between lenses and cornea and iris. 2. Other eye diseases, injuries, cornea, or iris diseases. 3. Uveitis, pain, itching, severe or continuous eye dryness. 4. Those diseases which can affect the eyes by wearing lenses. 5. Allergic reaction to lenses. 6. A systemic disease which can affect the eyeball, or a possibility that wearing lenses could worsen a systemic disease. 7. A condition which makes compliance with the relevant regulations for use of lenses impossible.

Manufacturer	USE BY
EC REP 1. EU representative 2. Sterilized 3. Sterile using steam or dry heat	4. CE Marking 5. Date of manufacture 6. Temperature limit

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