EN Patient insert
Please read carefully and keep this package insert for future reference. Peroxide Solution disinfects, removes protein & stores all soft contact lenses, including silicone hydrogels.

CONTENTS:
Sterile and buffered 3% hydrogen peroxide solution with a phosphonic acid stabilizer, sodium chloride and phosphate buffers.

DIRECTIONS FOR USE
- Always wash and dry your hands before handling your lenses and after using Peroxide Solution (see image A).
- Check that the special lens case includes the neutralising disc (see image B).
- Place your left and right lenses on the domes inside the appropriate baskets. Close the baskets taking care not to trap the lenses (see image C).

- Hold the lens case lid and rinse the lenses in the baskets with Peroxide Solution for 5 seconds each, over a sink/basin (see image D).
- Fill the lens case up to the fill line with fresh Peroxide Solution then immediately place the holder basket into the lens case. Do not invert or shake case (see image E).
- Your lenses will be ready to wear after soaking for 6 hours (see image F).
- Once lenses are removed from the baskets, discard any remaining solution from the lens case, rinse the case with Peroxide Solution over a sink/basin, and leave the lid off the case to air dry (see image G). If you do not intend to wear your lenses immediately after the disinfection/neutralisation process, repeat the steps above after 24 hours and before inserting your lenses. If, as a precautionary measure, be sure to rinse the lenses thoroughly with saline before soaking in your contact lens case.

WARNINGS
Only use the enclosed special lens case and neutralising disc with Peroxide Solution. DO NOT USE WITH OTHER LENS CASES (see image 1). NEVER insert Peroxide Solution directly into the eye (see image 2). DO NOT place your lens into the eye until complete neutralisation of the solution has occurred (minimum 6 hours). Do not rinse lenses with Peroxide Solution prior to inserting lenses into your eyes. If you want to rinse your lenses, use a sterile saline solution (see image 3).

Contraindications (reasons not to use):
If you are allergic to any ingredient in this product, do not use. Failure to discard solution from lens case after each use, topping off solution or use of water to case for your lenses may lead to contamination of lenses in eye injury and potential loss of vision. See accompanying instructions for additional important safety information.

Precautions:
Always use discard solution from lens case after each use. Keep the bottle tightly closed when not in use. Use before expiration date printed on the carton and bottle. Discard bottle and lens case 90 days after opening or after 35 uses, whichever occurs first. Store at room temperature. Always use the solution at room temperature. Do not use if tamper evident seal is missing or broken. Only use solution with specially designed lens case. Do not invert or shake case as residual unneutralised peroxide could harm the eye. If you experience any eye irritation or discomfort in this product, do not use. **KEEP OUT OF REACH OF CHILDREN.**

Important Safety Information:
Always follow the product directions for use. Failure to follow product directions may lead to vision loss or will result in burning and stinging. Peroxide Solution is NOT a multi-purpose solution. Do not use Peroxide Solution or the lens directly into the eye until complete neutralisation of the solution has occurred (minimum 6 hours). Do not use Peroxide Solution prior to inserting lenses into your eyes. If you want to rinse your lenses, use a sterile saline solution. Always wash and dry hands before handling lenses. Do not use tap water, bottled water or saliva with lenses or lens case. Only use fresh solution to clean and disinfect contact lenses. Always replace your solutions, lenses and lens case as directed. To avoid contamination, do not touch tip of container to any surface. Replace cap after rinsing. Not for use with heat (thermal) disinfection. Visit your eye care practitioner regularly.

How supplied: Peroxide Solution is available for purchase in 120 and 360 ml plastic bottles and supplied with a new period lens case. Bottles and cartons are marked with a lot number and expiration date.

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