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EN Eye lid cleansing wipes for eye lids and lashes.

To thoroughly but gently cleanse, hydrate and calm. To cleanse the eye area of ocular secretions, sticky residue and scales. Cleansing wipes are suitable for children, adults and contact lens wearers. Cleansing wipes adjusted to the pH of tears.

Ingredients:

WATER, GLYCERIN, PENTYLENE GLYCOL, TRIDECETH-8, CENTAUREA CYANUS WATER, EUPHRASIA OFFICINALIS FLOWER/LEAF/STEM WATER, SODIUM CHLORIDE, HYDROXY-ACETOPHENONE, PEG-40 HYDROGENATED CASTOR OIL, PHENOXYETHANOL, DISODIUM PHOSPHATE, DISODIUM EDTA, BISABOLLOL, SODIUM PHOSPHATE.

Instructions for use:

- Tear open foil pack, unfold wipe.
- Close your eye and thoroughly but gently cleanse the eye lids and lashes.
- Use a fresh wipe for each eye.
- Discard the wipe after use. Rinsing is not necessary.

Use as warm compress: put the unopened sachet for a couple of minutes in warm water. Before use test the temperature. Use as cold compress: put the unopened sachet for a couple of minutes in the refrigerator. Attention: do not put in the freezer.

Warnings:

- Do not apply directly in the eye.
- Do not use if sachet is opened or damaged.
- For external use only.
- For single use.
- In case of eye irritation, discontinue use and consult your doctor/contact lens specialist.
- Keep out of reach of children.
- Protect from sunlight.
- Store between 4°C and 25°C (40°F and 77°F).

Side Effects:

Do not use Eyelid Wipes if you are allergic to one of the ingredients.

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